




RAPID PAIN RELIEF



MICROBIAL BARRIER



WATERPROOF

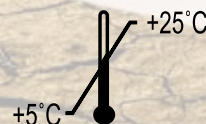

AMBIENT STORAGE

# Effective heel crack treatment in one simple step...

**Podiatric Care Innovation**  
from Bailey Instruments Ltd

**BAILEY**  
INSTRUMENTS LTD.



**LiquiHeel™**  
**Heel Crack Treatment**  
Sterile Topical Tissue Adhesive



[www.baileyinstruments.co.uk](http://www.baileyinstruments.co.uk)

helpline +44 (0) 161 860 5849

email | [sales@baileyinstruments.co.uk](mailto:sales@baileyinstruments.co.uk)

**LiquiHeel™** rapidly polymerises on contact with the skin to form a durable, flexible and waterproof, microbial barrier dressing. Clinical studies\* have shown that this offers rapid and long lasting pain relief and forms a moist wound healing environment that may aid the healing of the heel fissure. **LiquiHeel™** is quick, simple and convenient to use. **LiquiHeel™** will naturally slough off after 7-10 days during re-epithelialisation of the skin.

## For the **NHS**

- **Reduced consultation/ treatment sessions**
- **Cost effective and high success rate against current treatments**

### Indications

**LiquiHeel™** is a sterile topical cyanoacrylate protective dressing product indicated for the treatment of Heel Cracks.

The Octyl-Blend 10™ monomer blend technology (10% 2-Octyl cyanoacrylate and 90% n-butyl cyanoacrylate) forms a protective treatment method. **LiquiHeel™** should be applied by trained podiatrists or nursing staff.


### Contraindications

- Single use device, do not use on multiple patients.
- Do not apply to internal organs, blood vessels, nerve tissue or mucous membranes.
- Do not apply to infected or chronic wounds.
- Do not use on patients with a known sensitivity to cyanoacrylate or formaldehyde.
- Do not use over flexor or extensor surfaces.



### Advantages for the Patient

- Instant and sustained pain relief
- Fast and effective treatment
- Waterproof
- Microbial barrier dressing

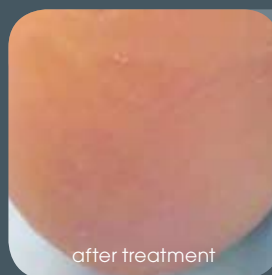

### Advantages for the Clinician

- Sets in <30 seconds
- Easy and quick to apply
- No secondary dressings required

## For the Private Podiatry Provider

- **Meets widely untreated patient needs**
- **Attracts new patients**
- **Supported by your practice marketing pack**

**twitter**

Join our **patient** discussion  
[www.twitter.com/LiquiHeel](http://www.twitter.com/LiquiHeel)

**facebook**

Join our **professional** discussion  
[www.facebook.com/pages/Liqui-Heel/108650505822827](http://www.facebook.com/pages/Liqui-Heel/108650505822827)

**BAILEY**  
INSTRUMENTS LTD.

\*Case series in private practice in co-operation with Southampton University 2009 SOCAP poster



**LiquiHeel™**  
**Heel Crack Treatment**  
 Sterile Topical Tissue Adhesive



[www.baileyinstruments.co.uk](http://www.baileyinstruments.co.uk)

**helpline +44 (0) 161 860 5849**

**email | sales@baileyinstruments.co.uk**

Manufactured in the UK by MedLogic Global Ltd PL7 5BG

